## Miracle Mix®

Prior to use, carefully read the instructions for use.

RADIOPAQUE SILVER ALLOY-GLASS IONOMER CEMENT RESTORATIVE

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

1. Core build-up
2. Restoration of primary teeth
3. Cases where radiopacity is required
4. As a base or liner.
5. Class I, limited Class II, temporary fillings, etc.

### CONTRAINDICATIONS

1. Pulp capping.
2. In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

#### 1. POWDER AND LIQUID DISPENSING

Pour the Miracle Mix alloy into the bottle of Miracle Mix powder and shake thoroughly.

- a) The recommended powder to liquid ratio is 2 or 3 level scoops of powder to 2 drops of liquid.
- b) For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- c) Hold the liquid bottle vertically and squeeze gently.
- d) Close bottles immediately after use.
- e) Shake the powder bottle occasionally.

#### 2. MIXING

Dispense powder and liquid onto pad or glass slab. Divide the powder into 2 equal parts. Mix the first portion with all of liquid for 15-20 seconds. Incorporate the remaining portion and mix the whole thoroughly for 20 seconds.

Note :

A heavy consistency is recommended.

#### 3. RESTORATIVE TECHNIQUE

- a) Prepare tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping, use calcium hydroxide.
- b) Wash and dry but DO NOT DESICCATE. GC CAVITY CONDITIONER or GC DENTIN CONDITIONER is recommended to remove the smear layer.
- c) Mix the required amount of cement. Working time is 1 min. 30 seconds at 23°C (73.4°F). Higher temperatures will shorten working time.
- d) Remove surface moisture but DO NOT DESICCATE.
- e) Transfer cement to the preparation using a syringe or suitable placement instrument. Avoid air bubbles.
- f) Form the contour.
- g) Setting time is 5'00" to 5'30" from start of mixing.

#### 4. FINISHING

Use standard techniques.

### STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life : powder 3 years, liquid 2 years)

### PACKAGES

1. Package : 15g powder with measure, 10g (8.0mL) liquid, 17g alloy.
2. Bottle of 15g powder with measure.
3. Bottle of 10g (8.0mL) liquid.
4. Bottle of 17g alloy.

### CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. DO NOT mix powder or liquid with any other glass ionomer components.

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3737 West 127th Street, Alsip, IL 60803 U.S.A.
**GC ASIA DENTAL PTE. LTD.**