

Exabite™ II NDS

Vinyl Polysiloxane
Bite Registration Material



- 132412 EXABITE II NDS Bite Registration Creme Two-Pack**
Contains: Two (50ml) cartridges and 12 mixing tips.
- 132418 EXABITE II NDS Bite Registration Creme Eight-Pack**
Contains: 4 two-packs of 50ml cartridges and 48 mixing tips
- 153248 Package of 48 Mixing Tips (Type 2L-Aqua)**
- 152101 GC Cartridge Dispenser 2**
New Delivery System NDS, fits all isolated barrel type cartridges

PHYSICAL PROPERTIES

Working Time (@23°C)*	45 seconds
Minimum time in mouth	45 seconds
Maximum Strain in compression*	0.9%
Recovery from deformation*	99.6
Linear dimensional change*	-0.10 %
Hardness Durometer	87 (Shore A)
Shelf Life	3 years

*Tested in accordance with ISO 4823:1992(E)

Advancing the Art and Science of Dentistry

GC Corporation	Tokyo, Japan	03-3558-5183
GC America Inc.	Alsip, IL U.S.A.	1-800-323-7063
GC Europe N.V.	Leuven, Belgium	+32-16-40-27-00
GC Asia Dental PTE. Ltd.	Singapore 508724	+65-5467588

Lit. Code 814001
© 1999 GC America Inc.

Exabite™ II NDS

Advanced Vinyl Polysiloxane Bite Registration Creme



Fast and Accurate Bite Registration



Exabite II™ NDS


Bite Registration Creme

Developed especially for occlusal registration, **Exabite II** is an improved vinyl polysiloxane material that has significant handling and accuracy advances, extrudes easy and sets hard very quickly.

In fact, it stacks better, flows better and sets 50% faster than original formula Exabite.

New Faster 45-Second Formulation


Exabite II provides an optimal placement time with a “snap” set feature. With a working time up to 45 seconds, the fast formulation sets in 45 seconds in the mouth. Less time in the mouth means less chance for bite distortion due to movement.

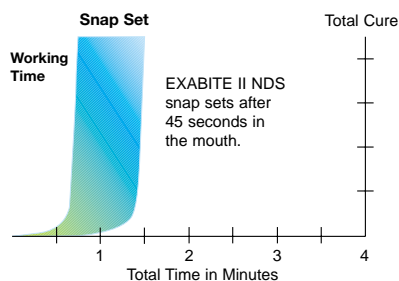

Extremely Hard Set

Exabite II sets hard for easy trimming and carving.

- 87 Plus durometer (Shore A)
- Dimensional Stability - less than 0.2% shrinkage @ 24 hours.
- Improved elasticity - removes easily from the mouth without distortion.
- Assures accurate articulation.

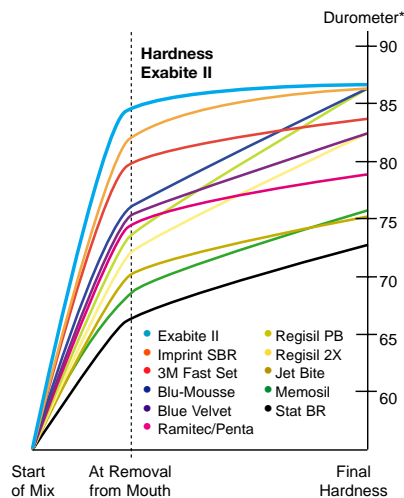

Vinyl Polysiloxane

No heat. No taste. No smell or odor. Your patients will remain comfortable and stress free.

Exabite II makes bite registrations fast and extremely accurate.




* Data compiled from published information and GC Test Data. Imprint SBR, 3M Fast Set, Blu-Mousse, Blue Velvet, Ramitec, Penta, Regisil PB, Regisil 2X, Jet Bite, Memosil and Stat BR are not trademarks of GC Corporation or its divisions and subsidiaries.



New Cartridge Delivery System NDS Easy-to-Use Automix Cartridges and Dispenser


The automix cartridge system assures trouble-free and bubble-free application. And the new GC Cartridge Dispenser 2 and isolated barrel cartridge design provides easier extrusion through the mixing tip, less chance of crossover contamination and tip clog.

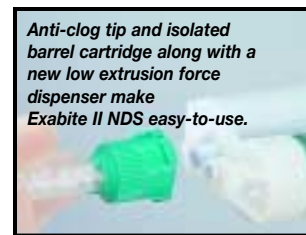

Anti-clog tip and isolated barrel cartridge along with a new low extrusion force dispenser make **Exabite II NDS easy-to-use.**

Simple Disinfection

Exabite II impressions can be disinfected by immersing in a 2.5% or 3.4% alkaline glutaraldehyde (for time recommended by glutaraldehyde manufacturer). Just rinse with water prior to and after soaking.

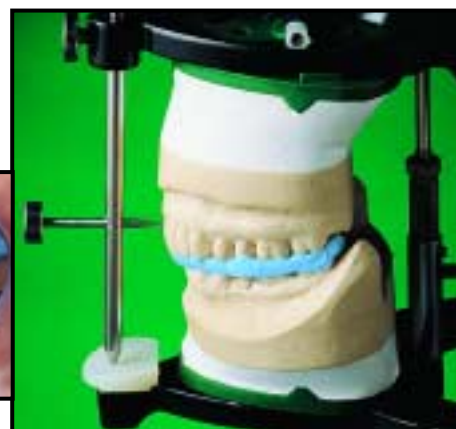

Try New **Exabite II NDS** and find out just how fast and accurate a bite registration can be.

Simple Technique

Fast and Accurate Bite Registration



Express **Exabite II** onto the occlusal surface. Working time is 45 seconds from start of extrusion.



Let patient occlude to normal bite. **Exabite II** flows easily under pressure and replicates accurate occlusal details.



Remove the bite registration from the mouth after 45 seconds.



Trim undercuts and excess material with a SHARP knife.



Check fit of bite registration on models and articulate case as necessary.