Prior to use, carefully read the instructions for use.

**EN**

## G-ænial™ SHADE GUIDE

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS  
Used for shade selection.

### HOW TO USE

- Clean the tooth with pumice and water. Shade selection should be made prior to isolation. Keep the teeth hydrated. Select the appropriate G-ænial shades by referring to the G-ænial shade guide.
- Each shade tab consists of four levels with 1.0, 1.5, 2.0 and 3.0 mm in thickness respectively. When layered restorations are desired, two or more shade tabs can be placed on top of each other to estimate the resulting layered shades.

### PACKAGE

1 piece

### NOTE

The shade guide can be cleaned with water and mild soap. It should not be sterilized. The use of various chemical disinfectants / sterilants may cause damage to the shade guide.

### Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:  
[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental) In this way you will contribute to improve the safety of this product.

Last revised: 06/2019

MANUFACTURED by  
GC DENTAL PRODUCTS CORP.  
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC ASIA DENTAL PTE. LTD.  
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541  
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.  
1753 Botany Rd, Banksmeadow, NSW 2019, Australia  
TEL: +61 2 9301 8200

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