Prior to use, carefully read the instructions for use. **EN**

## GC Fuji™ II CAPSULE

RADIOPAQUE GLASS IONOMER RESTORATIVE CEMENT IN CAPSULES  
For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

- Restoration of primary teeth.
- Core build-up.
- Restorations of Class III, V and limited Class I cavities.

### CONTRAINDICATIONS


- Pulp capping.
- Avoid use of this product in patients with known allergies to glass ionomer cement.

### DIRECTIONS FOR USE

Powder / Liquid Ratio (g/g)	0,30 / 0,11
Mixing Time (sec.)	10"
Working Time (23°C / 73°F) (min., sec.)	2'00"
Net Setting Time (37°C / 99°F) (min., sec.)	2'20"
Finishing Time (37°C/99°F, from start of mixing) (min.,sec.)	15'00"

### 1. CAPSULE ACTIVATION AND MIXING


- Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 1).
- To activate the capsule, push the plunger until it is fully depressed with the main body and hold it down for 2 seconds (Fig. 2).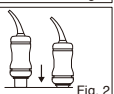

- Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder and liquid.
- The capsule should be activated just before mixing and used immediately.

### 2. RESTORATIVE TECHNIQUE

- Prepare tooth using standard techniques. Extensive mechanical retention is not necessary.
- Wash and dry but DO NOT DESICCATE. CAVITY CONDITIONER or DENTIN CONDITIONER is recommended to remove the smear layer. For pulp capping use calcium hydroxide.
- Mix the cement. Working time is 2 minutes from the start of mixing at 23°C (73°F). Higher temperatures will shorten working time.
- Immediately remove the mixed capsule from the mixer and load it into the GC CAPSULE APPLIER.
- Make two clicks to prime the capsule then syringe (Fig. 4).
- Remove surface moisture, but DO NOT DESICCATE.
- Extrude cement directly into preparation. Avoid air bubbles.
  - To adjust the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.
  - To remove the used capsule, push the applicator release button. Twist the capsule and pull upwards.
- Form the preliminary contour and cover with a matrix if required.
- Once set, remove the matrix and immediately apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure).

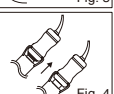### 3. FINISHING

- One visit treatment
  - After initial set, correct the contour WITHOUT WATER SPRAY using standard techniques.
  - Apply GC Fuji VARNISH or GC Fuji COAT LC and wait for about 15 minutes.
  - Final finishing and polishing under water spray can now be carried out.
  - Apply a final coat of GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure for 10 sec.) or G-COAT PLUS (light cure for 20 sec.)
- Standard method
  - 24 hours after placement finish and polish the restoration under water spray using standard techniques.

### STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F).

### SHADE

No. 21	pale yellow	No. 23	dark grey
No. 22	yellow brown	No. 23M	deep grey
No. 22.5	grey brown	No. 26	deep brown

### PACKAGES

- Assortment package  
50 capsules (5 pcs. No. 21, 26, 10 pcs. No. 22, 22.5, 23, 23M)
- Single shade package: 50 capsules  
\* Average contents per capsule: 0.30 g powder and 0.11 g (0.09mL) liquid  
\* Minimum net volume of mixed cement per capsule: 0.12 mL  
Option: GC CAPSULE APPLIER (1 piece)

### CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

- <http://www.gceurope.com>
  - or for The Americas <http://www.gcamerica.com>
- They can also be obtained from your supplier.

Last revised: 06/2019

Rx Only

MANUFACTURED by  
**GC CORPORATION**  
761 Hisatsuna-cho, Tabashi-ku, Tokyo 174-8585, Japan

**EU : GC EUROPE N.V.**  
Polderdijk 66, Herestraat 49, 3000 Leuven, Belgium  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

RESPONSIBLE MANUFACTURER IN CANADA:  
**GC AMERICA INC.**  
3727 West 127th Street, Aliso, IL 60803 U.S.A.

DISTRIBUTED by  
**GC CORPORATION**  
761 Hisatsuna-cho, Tabashi-ku, Tokyo 174-8585, Japan  
**GC AMERICA INC.**  
3727 West 127th Street, Aliso, IL 60803 U.S.A.  
TEL: +1-708-567-0900  
[www.gcamerica.com](http://www.gcamerica.com)

**GC SOUTH AMERICA**  
Rua Heliodora, 399, Santana - São Paulo, SP, BRASIL  
CEP: 05002-001 - TEL: +55-11-2925-0965  
CNPJ: 08.278.999/0001-61  
RESP. TEC: Mayara de Sales Ribeiro - CRO/SP 105.582

**GC ASIA DENTAL PTE. LTD.**  
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541  
TEL: +65 6546 7588

**GC AUSTRALASIA DENTAL PTY. LTD.**  
1753 Botany Rd, Bankmeadow, NSW 2019, Australia  
TEL: +61 2 9301 8200

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