

Glass Hybrid Restorative System Technique Guide



1. Occlusal caries.



2. Cavity preparation.



3. Apply CAVITY CONDITIONER (10 sec.) or DENTIN CONDITIONER (20 sec.)



4. Rinse, dry; do not desiccate.



5. EQUIA Forte® HT Fil is placed in one increment.



6. Shape and contour within 60 sec.



7. Final finish under water spray after 2min 30 sec.*



8. Brush-apply a single layer of EQUIA Forte® Coat.



9. Light-cure for 20 sec.



10. Completed restoration.

* During the first 2 min 30 sec. from start of mix extra care should be taken to avoid moisture contamination or drying-out. In case this cannot be guaranteed, immediately apply EQUIA Forte® Coat and light cure.



+



=

Glass Hybrid Restorative System

EQUIA Forte® HT Fil Refill

Contains: 50 EQUIA Forte® HT Capsules

Shades: A1, A2, A3, A3.5, B1, B2, B3, C4

Assorted (contains 10 each, A2, A3, A3.5, B1, B3)

EQUIA Forte® Coat

Bottle 4mL

Also available EQUIA Forte® Fil



- Bulk placement in less than 5 minutes from start to finish
- No bonding agent needed – chemical adhesion with tooth structure
- Low moisture sensitivity
- Optimal protection of marginal seal for long lasting restorations
- Great shade-matching ability
- High lustre, translucent coating, saving you polishing time



GC Australasia Dental Pty Ltd
1753 Botany Road Banksmeadow
NSW 2019 Australia

T: +61 2 9301 8200
E: info@gcaustralasia.com
www.gcaustralasia.com